

User Guide

With AmiCane, venture out yet remain connected with your loved ones



Know your Amicane

All is Well Button

Press this button when you trigger alert by mistake

SOS button

Press this button when you are in distress

LED Torch

You can press this button to change intensity of the light.

Fall Sensor

Stick has inbuilt orientation sensor which is used to detect if stick is fallen or not

Charging Port

Use Standard Type-C connector

Charging LED

Amicane companion Android app

There are two types of users

User



Amicane user must download and configure companion app


Caregiver



At least one caregiver must download and configure companion app



Scan to download the app

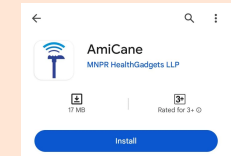

Click on Install button

2

4

Introduction

Welcome to the AmiCane Smart Walking Stick.

This user manual will guide you through the features and usage of your AmiCane. Please read it carefully before use to ensure a safe and enjoyable experience



Adjust Amicane Height

1 Unlock the safety ring



2 Press silver button and adjust height




3 Lock the safety ring



Amicane Serial Number

Keep this QR safe
Do not misplace this

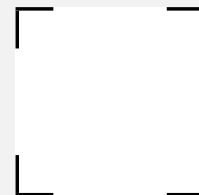

Log into Android application

Log into app using mobile number and OTP

User





Use mobile number of User to log into app, this will be used to initiate phone calls when user needs help

Caregiver



Use mobile number of Caregiver to log into app, this will be used to receive phone calls when user needs help

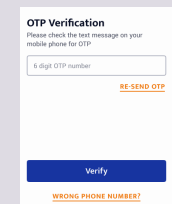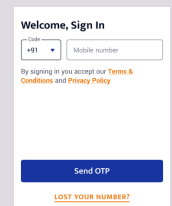

1

3

5

Amicane user onboarding

Preparing your amicane for onboarding

- Charge your stick for at least an hour before starting onboarding process

- Press SOS (Orange colour) button and LED (White colour) button



simultaneously for at least 10 seconds

You should hear 4 beeps from stick, If you hear 2 beeps (or no beep) repeat the process once again




Ensure that stick is working by operating the LED torch



- Wait for application to establish a connection with Amicane, once connected you should see a flash of light fading out on Amicane
- Fill up user information, this will be used as ID card in case of emergency, hence provide the correct information

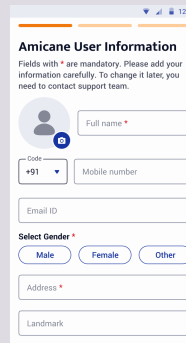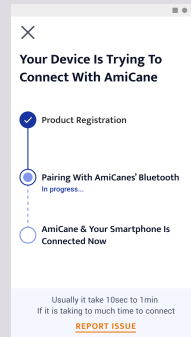

- Enter details/Information of caregiver
- You will be asked to enter/verify the information about Amicane User, please review the information carefully
- Setup Geofence center location and radius

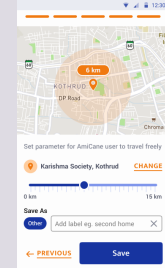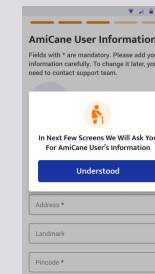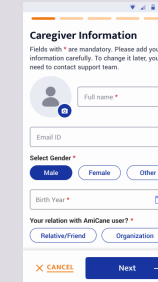



6

8

10

User onboarding process in application

- Install and open Amicane app, provide all permissions asked by the application prompt
- Log into Amicane app using **user's** mobile number and OTP sent on the number
- Tap "**AmicaneUser**" when asked
- Scan QR code of Amicane Serial Number (See Page 3) or enter Serial Number manually

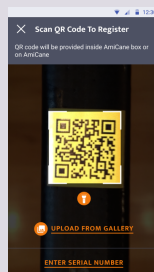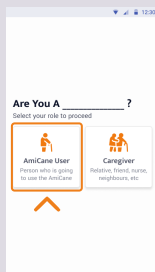

Amicane caregiver onboarding

- Install and open Amicane app, provide all permissions asked by the application prompt
- Log into Amicane app using **caregivers's** mobile number and OTP sent on the number
- Tap "**Caregiver**" when asked
- Scan QR code of Amicane Serial Number (See Page 3) or enter Serial Number manually



Safety guideline

- Keep the Amicane away from water and heavy temperatures
- Do not disassemble or modify the Amicane as it will lead to malfunctions or accidents
- Use good quality chargers to charge the Amicane

For more information visit <https://www.amicane.in/>

7

9

11